


TIVA sets

TIVA Set 2 (V86, dec.MF-2, 2xFLL, MLL)

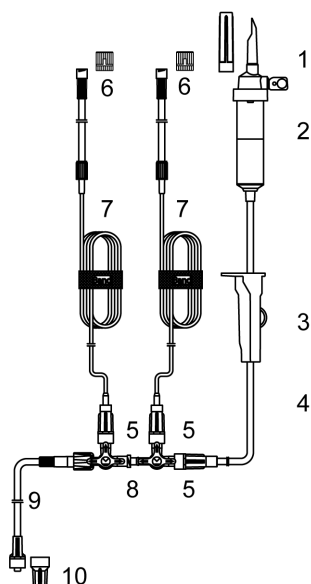

1. The special design of the spike enables easy penetration of container ports and offers a safe connection to the container. Vented version. The protection cap maintains sterility of the set.
2. Air vent with hydrophobic, sterile air filter, closure cap and additional CODAN Check Valve in the airway, protects the filter against wetness which would cause filter blockage. Standard drip tube (20 drops/1 ml). The flexible, transparent non-reflective material with level ring mark enables easy priming and control of drip rate. Equipped with 15 micron fluid filter.
3. Precision roller clamp enables exact adjustment of the desired flow rate maintaining it with minimum variation. Additionally, it is equipped with a clamp/hanging device for tubing and for protection against stick accidents after use of the set.
4. Transparent, flexible PVC tubing with plasticiser TOTM 3,0/4,1 mm (ID/OD) in the set (regular bore).
5. Male Luer-Lock adapter with integrated CODAN Check Valve prevents any backflow of solution.
6. Female Luer-Lock adapter guarantees a safe connection.
7. Transparent, flexible PVC tubing with plasticiser TOTM 0,9/2,0 mm (ID/OD) (mini bore).
8. Disconnectable 3-way stopcock manifold with different coloured stopcock and various flow way adjustments enables simultaneous administration of medication with e.g. set or syringe.
9. Transparent, flexible and kink-resistant PVC tubing with plasticiser TOTM 2,5/4,1 mm (ID/OD) (regular bore).
10. Male Luer-Lock adapter guarantees a safe connection to the venipuncture device. The FlowStop protection cap for Male Luer-Lock adapters with integrated hydrophobic membrane ensures internal sterility prior to use. During priming, the FlowStop protection cap remains attached to the Male Luer-Lock adapter and allows air to pass but as soon as solution reaches the membrane, the flow stops automatically.



Disclaimer:

The above is indicated only for information. This document is not a specification. The values given are subject to the usual fluctuations. Please follow the instructions for use by the medication manufacturer and the actual SPC (Summary of Product Characteristics). Common hygiene measures and working methods are to be followed at all times.

Subject to alterations without prior notice!

REF	Product description	Packaging	GTIN	Filling/Storage volume
76.4012	V86, 175 cm PVC (TOTM) tubing 3,0/4,1 mm (ID/OD), MLL adapter with integrated CODAN Check Valve, 2x 150 cm PVC (TOTM) extension line 0,9/2,0 mm (ID/OD), disconnectable MF-2, 15 cm PVC (TOTM) tubing 2,5/4,1 mm (ID/OD), MLL adapter, FlowStop	1 40 1280	04055698004843 04055698036967 -	15 ml (40°C) 18 ml (40°C/2 bar)