Español

Uso previsto

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English

Intended Use

During manual or mechanical ventilation, the mask is used to seal the mouth and nose area.

Contraindications

For non-fasting patients and increased risk of aspiration, the mask ventilation should not be performed, except in emergency situations.

Safety Instructions

- Insufficient ventilation can cause permanent brain damage to the patient.
- Properly seal the mask around the patient's face to ensure adequate ventilation.
- Maintain the correct head tilt to keep the airway open! In case of cervical spine injury check if the head should be tilted or not.
- If adequate ventilation is not possible with this product, start immediately with mouth-to-mouth respectively mouth-to-nose ventilation.

Instruction for use

1. Check the mask for leaks in the packaging by pressing with the hand on the cushion.
2. Apply the mask firmly to the patient's face to achieve a tight seal. If necessary, inflate or deflate the cushion by using a syringe.
3. Hold the mask with hand or secure with a head strap.

ATTENTION: The face mask is a single use product and therefore cannot be reused! Further recommendations for our single use products are available at www.vbm-medical.de in the download section.

Storage Conditions

On the following conditions our products can be stored at least for five years:

- Beware of direct solar radiation and artificial light
- Do not store silicone products with PVC or rubber products

Disposal

Used or damaged products must be sterilised before disposal. Our products can be disposed according to the valid legal regulations.

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