

Installation Guide

GCX Channel Mounted 12"/30.5cm 10x25mm Rail
(Refer to qualified personnel)


The purpose of this guide is to:

1. Describe rail position adjustment
2. Describe attachment of Rail Assembly to channel
3. Reference examples of Rail mounted accessories

Parts Reference

The following parts and hardware are included in this installation kit:

Item #	Description	Qty
1	Rail Assembly	1
2	1/8" Hex Wrench (not shown)	1
3	M4 Hex Wrench (not shown)	1


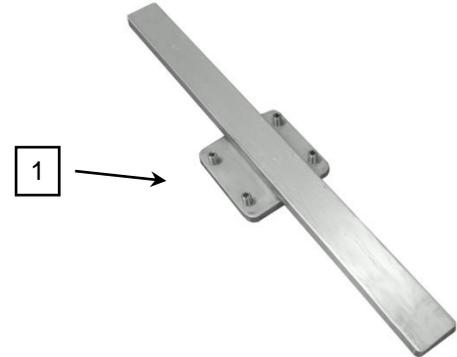


Tools required

- 1/8" hex wrench (provided)
- M4 hex wrench (provided)



Rail Position Adjustment

- A. Using M4 Hex Wrench provided, remove two (2) M6 x 25mm FHSCS to separate rail from slide assembly.
- B. Adjust rail to desired position and install two (2) M6 x 25mm FHSCS that were previously removed.

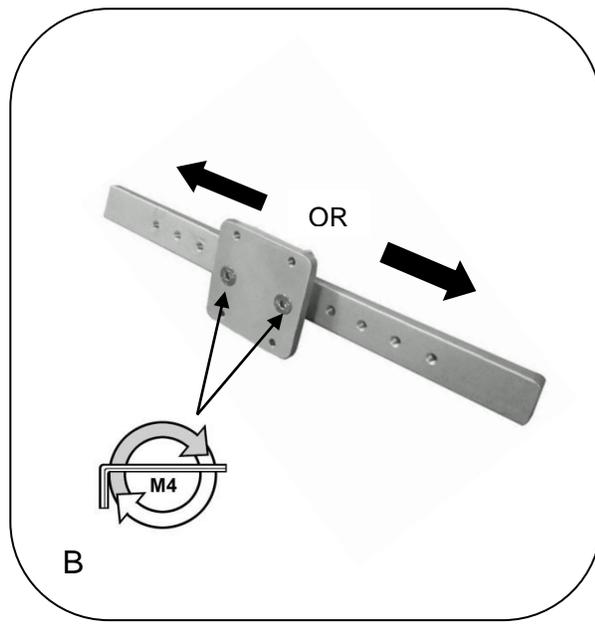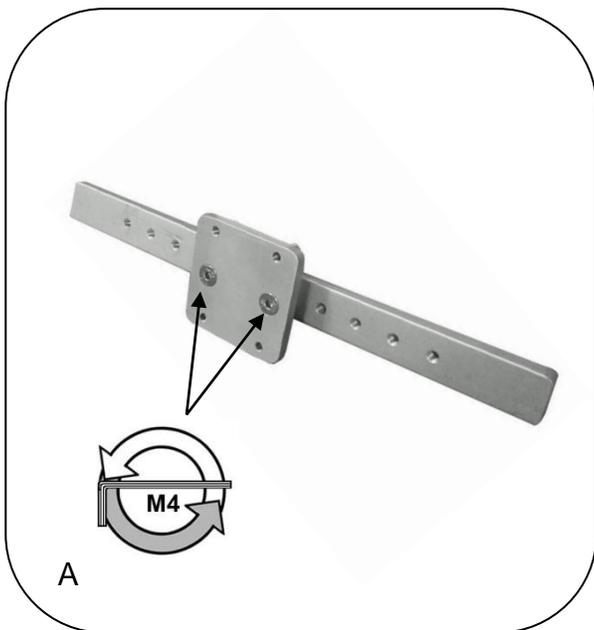



Installing Rail Assembly in Channel

- A. Guide the assembly into the top of the channel.
- B. Slide to desired position in the channel and tighten all four (4) set screws using the 1/8" hex wrench provided.

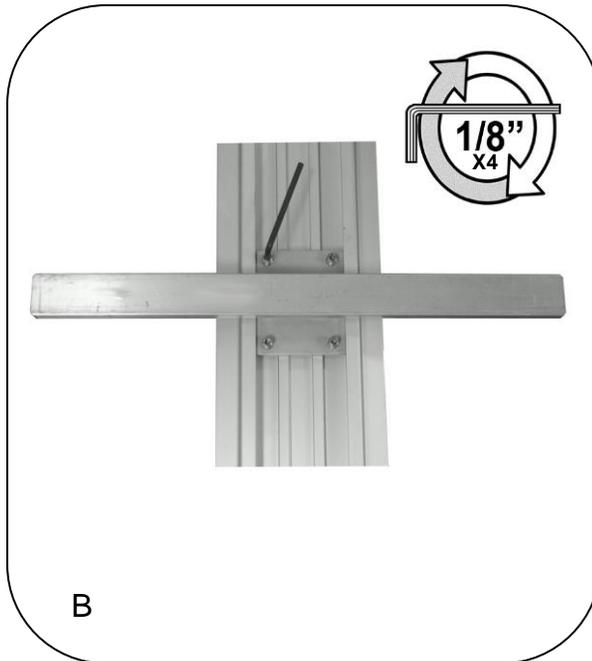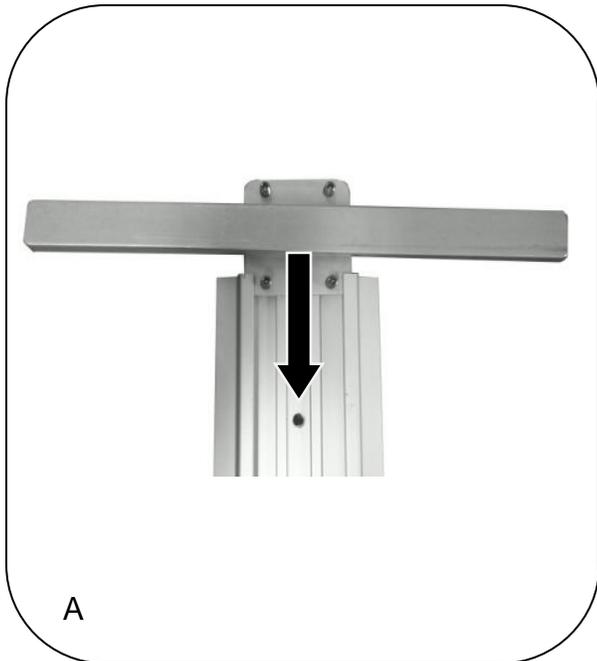

Examples of Rail Mounted Accessories

Accessories mounted on this assembly must not exceed 20lbs/9.1kg.

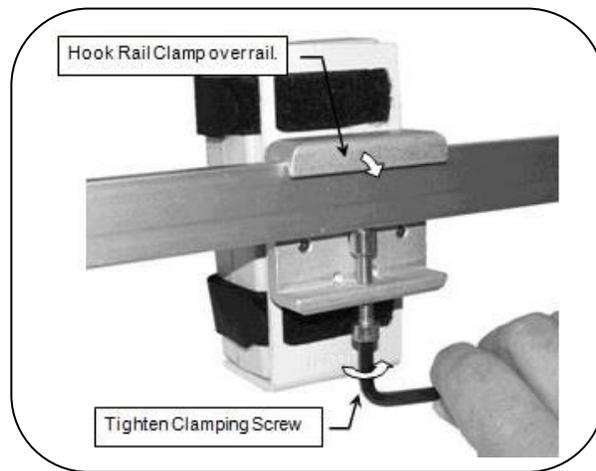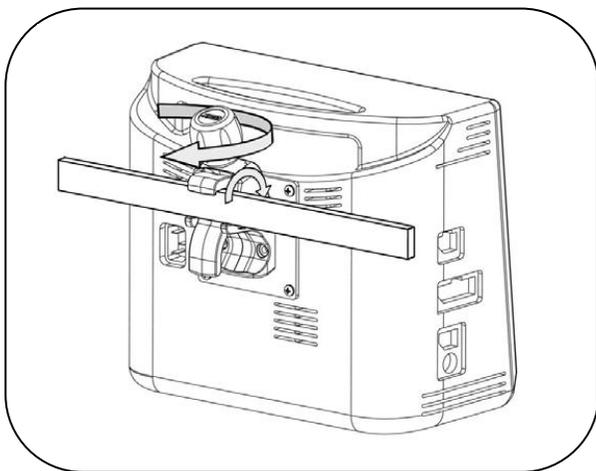

Routine Maintenance

Periodically inspect all fasteners associated with the mounting system. Tighten or adjust fasteners as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

1. The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
2. The surface finish will be permanently damaged by strong chemicals and solvents such as acetone or trichloroethylene.
3. Do not use Steel wool or other abrasive material.
4. Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test any cleaning solution on a small area of the mounting assembly that is not visible to verify compatibility.
5. Never submerge or allow liquids to enter the mounting assembly. Wipe any cleaning agents off of the mounting assembly immediately, using water dampened cloth. Dry the assembly thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessory equipment, refer to the specific instructions delivered with those products before installing the arm in the Channel.