Verwendung

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Use

For dosing oxygen, compressed air and other medical gases; in connection with humidifiers or atomizers for vaporizing and atomizing.

Accessories:

- humidifiers
- atomizers
- one-way humidifiers
- tube-connection 4/6 mm

Operation

Prior to coupling the plug into the gas outlet, set the latch flow meter to "0". Fasten rail units on the medical instruments rail. For plug-in units, plug the unit into the gas outlet; for rail-units, plug the angle plug into the gas outlet.

Always connect accessories by hand: prior to first use, completely sterilize humidifier and atomizer; screw tube-connection and, respectively, humidifier/atomizer under the flow meter; fill reusable bottle with sterile water up to the mark and screw under the bottle head; connect additional accessories.

Set the desired gas volume via the hand wheel. During use, pay attention to a sufficient water level of humidifier or, respectively, atomizer.

After use, reset latch flow meter to "0". The flow meter can remain in the outlet, preferably in park position. If required, separate humidifier or atomizer from the flow meter, drain and sterilize bottle.

Additional information

- Always keep all connections free from oil and grease!
- Never use tools when connecting the unit and the accessories; all connections can be tightened or loosened by hand.
- The Kolibri may only be used for the purpose outlined in this instruction of use and by briefed staff.
- No flow between two flow steps!
- Subject to technical change without notice; other designs upon request.
- Fluctuations with pre or counter-pressure, as well as the site conditions may have influence on the flow rate.
- There is no warranty for correct function in case of improper transport or storage or during operating the Kolibri not in accordance to this instruction of use.

Technical data

Type: latch-down flow rate adjustment for medical gases, indication via flow figures, chrome-plated metal design as a plug-unit or as a rail-unit

Capacity: adjustable flow rate in steps between:

Flow step	1	2	3	4	5	6	7	8	9	10
Neonatal	0,1	0,15	0,2	0,25	0,3	0,4	0,5	0,6	0,8	1 l/min
Children	0,1	0,3	0,5	0,6	0,8	1	2	3	4	5 l/min
Standard	0,5	1	1,5	2	3	4	6	8	12	15 l/min
Standard+	0,5	1	2	4	6	8	10	12	15	30+ l/min

Accuracy of flow: $\pm 20\%$ of measuring value;
for measuring value $\leq 1,5$ l/min ± 30 of measuring value

Inlet: gas-type specific plug inserts according to DIN 13260-2
optional inserts for e.g. SS 8752430, NF S90-116, BS 5682
rail units: NIST (DIN EN ISO 5359)

Outlet: 9/16-18 UNF (for standard one-way humidifiers)

Environment-condition: storage -30 ... + 50 °Celsius
use: 0 ... + 40 °Celsius

Dimension(DIN): plug-unit: ca. 109x62x40 [LxHxW in mm]
rail-unit: ca. 90x105x40 [LxHxW in mm]

Guideline: DIN EN ISO 15002
DIN EN ISO 5359
DIN 13260-2

Medical Products Directive 93/42/ECC; Classification: IIa

Maintenance/Service

External disinfection with a cloth is sufficient for all chrome-plated brass parts (with customary cleaner). Sterilize humidifier and atomizer bottle at max. 134° C. Let equipment dry completely before using it again.

Prior to each use, make a visual check of flow meter, humidifier and atomizer (external damages). The units should have a maintenance check at least once every five years (function and leak test), if the flowmeter is used to an unusually high degree, the intervals between maintenance might be shorter. Failures and damages may only be remedied by authorized expert personnel. Use only original **GREGGERSEN** spare parts!

Spare parts

pos	number	name
1	900619	tube-connection 9/16

1