



## Laryngoscope Handle with Natural Daylight LED



### Each GreenLED™ handle contains:

- Green outer silicon (type material) sleeve
- Inner cartridge with screw-on cap
- Not supplied: AA Alkaline Batteries

Description	Ordering Code
GreenLED™ handle	4215
Alkaline Batteries	Not supplied

### Battery placement in GreenLED™ Handles

- Remove handle cartridge from handle body by opening black bayonet cap at bottom of handle and unscrewing cap of the cartridge
- Insert 2 new alkaline "AA" size batteries into the cartridge ensuring correct polarity orientation of the batteries ("+" side first).
- Screw cap to cartridge.
- Insert the handle cartridge into the body of the handle.
- Lock on the black bayonet cap really tight.
- Press finger onto LED light source and test light performance.
- Discard used or defective batteries in an appropriate waste disposal receptacle.

*Do not incinerate batteries.*

### ROUTINE CARE AND MAINTENANCE

GreenLED™ handles should always be cleaned and thoroughly dried before storing in a dry place.

Take care not to scratch the LED light source.

Batteries should be removed from handles being stored for periods of more than 7 days to avoid possible leakage of corrosive materials from the batteries.

### CLEANING, DISINFECTION AND STERILIZATION

ALWAYS COMPLY WITH LOCAL HEALTH & SAFETY REGULATIONS. WHEN USING DISINFECTANT OR STERILISING SOLUTIONS, ALWAYS OBSERVE MANUFACTURERS' RECOMMENDATIONS FOR PROCESS CONCENTRATIONS AND TIMES.

### Cleaning and Disinfecting:

- Immediately after use, clean and disinfect all outer surfaces using a soft cloth dampened with soapy water, alcohol or disinfectant solutions.

- Rinse and dry thoroughly both the inside and outside of the handle barrel before replacing batteries or putting into storage.

**Sterilization:** Outer handle (silicon type material) may be autoclaved according to manufacturer's recommended temperatures and time instructions

GreenLED™ handles are suitable for treatment in thermal disinfectors (with cartridge removed) and for sterilization using Ethylene Oxide (ETO), Steris® and Sterrad® sterilization procedures.

*Do not autoclave inner cartridge.*

**Cleaning Of Inner Cartridge:** Clean the inner cartridge of the GreenLED™ handle by carefully wiping its outer surface using a low level surface disinfectant following the instructions of the manufacturer regarding how to apply. On no account should this inner core be immersed in any type of solutions or be exposed to temperatures above room temperature .

**LED light source:** The LED light source does not have to be replaced and is fully guaranteed for 5 years from date of manufacture.

**Warning:** Only Alkaline AA disposable batteries should be used. Lithium Iron Disulphide and all re-chargeable batteries should not be used with GreenLED™ handles and will lead to destruction of the LED illumination.

Thank you for purchasing Truphatek products. Proper care, maintenance and the use of Truphatek spare parts, accessories and bulbs will ensure the long life and satisfactory performance of your purchase.

Whilst every care is taken to ensure the accuracy of these instructions, we reserve the right to make modifications and improvements as we deem necessary in the interest of our customers.

Truphatek laryngoscope blades and handles carry a 5 year warranty on all stainless steel parts against defects in manufacture and materials.

Full text of the Truphatek warranty and further information about Truphatek products are available on our website [www.truphatek.com](http://www.truphatek.com) or through our Customer Service desk at [helpdesk@truphatek.com](mailto:helpdesk@truphatek.com).

Note: Truphatek™ and GreenLED™ are trademarks of Truphatek International Ltd. and are not to be used without written permission.

Compatible with all Green System laryngoscope blades.

Conforms to ISO 7376:2009

Approvals : ISO 9001: 2008, ISO 13485:2003

FDA registered manufacturer



Manufactured by Truphatek International Ltd  
P.O. Box 8051, Netanya, Israel, 42504  
Telephone: +972-9-8851155 Fax: +972-9-8851212  
[helpdesk@truphatek.com](mailto:helpdesk@truphatek.com) [www.truphatek.com](http://www.truphatek.com)  
2011©Truphatek® All rights reserved.

42150001 Rev 1-07/12