



Laryngoscope Handle with Natural Daylight LED

**Natural Daylight Spectrum  
reproduces colors faithfully,  
providing clear and  
accurate color contrasts**

**Sterrad  
Compatible**



### **A Light to Rely On**

- Long lasting, reliable LED with over 10,000 hours of uniform, consistent light
- 2000 LUX for optimal visualization during intubation
- Ergonomic, non-slip thermoplastic elastomer grip for maximum comfort
- Durable stainless steel construction; fully autoclavable
- Unique, patented design for superior performance
- Compatible with all Green System blades
- Powered by two AA alkaline batteries
- 5 year warranty

Compatible with all Green System laryngoscope blades  
Conforms to ISO 7376:2009  
Approvals: ISO 9001:2008, ISO 13485:2003  
FDA registered manufacturer



[www.truphatek.com](http://www.truphatek.com)

# Maximize your blade performance with our premium grade handle

## Why is Truphatek’s LED source superior?

Truphatek has optimized the color temperature of its LED source to produce a light spectrum that corresponds to natural daylight. With a high Color Rendering Index (CRI), the GreenLED provides clear and accurate color contrasts for superior visualization during intubation.

## Does the LED need to be replaced from time to time?

No, the LED is fully guaranteed by Truphatek for 5 years from date of manufacture.

## How many intubations can be performed before replacing the batteries?

No less than 300 intubations can be performed with one pair of new alkaline AA batteries.

## What kind of batteries may be used with the GreenLED handle?

Only alkaline AA disposable batteries. Lithium Iron Disulphide and all re-chargeable batteries should *not* be used with the GreenLED handle.

## How can I clean the GreenLED handle?

The outer sleeve is fully autoclavable when removed from the inner cartridge. The inner cartridge should not be autoclaved; wipe it gently with a dry cloth, taking care not to scratch the LED.

## Product Specifications:

Ordering Code	Product	Description
4215	GreenLED Handle	GreenLED ergonomic handle with battery cartridge
	Handle and Blade sets upon request	Sets of 3 or 4 Macintosh or Miller GreenSpec™ blades, with one GreenLED handle and battery cartridge

## About Truphatek:

Established in 1987, Truphatek is a leading provider of airway management and emergency medical devices. Our mission is to develop a steady stream of novel, patentable medical devices for global marketing. High-tech design is matched with low cost and high quality production capabilities. The company’s extensive line of disposable and reusable laryngoscopy products have a world class reputation.



### From Truphatek’s Warranty Statement

All Truphatek products are unconditionally guaranteed to be of good quality and free from manufacturing defects in material and workmanship for a period of 5 years from date of manufacture. Replacement product will be provided without charge for any Truphatek product not meeting this warranty.

Full text of Truphatek’s warranty statement can be viewed at our website [www.truphatek.com](http://www.truphatek.com) or by request from Customer Service at [helpdesk@truphatek.com](mailto:helpdesk@truphatek.com).

GreenLED, GreenSpec and Truphatek are trademarks of Truphatek International Ltd. and should not be used without prior written permission.

Distributed by