


## Placement of PIC Securing Strap on BIS Monitoring System

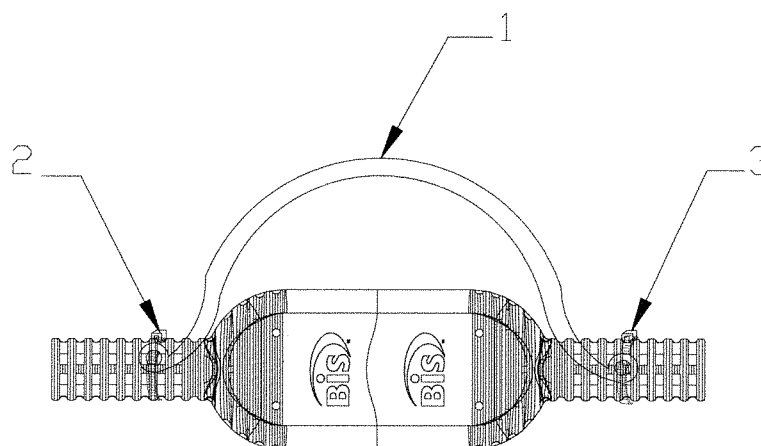

1. While the PIC PLUS is connected to the DSC cable, place the loop strap over the strain relief as indicated.
2. Insert each cable tie through the orifice at the end of the loop and secure the tie firmly by pulling it through its own securing latch. Cut the excess tie so that the strap is flush to the latch and no excess material is exposed.
3. Repeat for the other side of the PIC mated cable.  
If cable has to be removed, do not cut the loop strap as it is intended to keep the system together.  
If either the loop strap or cable tie are cut for whatever reason, please replace as soon as possible.

Loop strap order number: 675-0015

Cable tie order number: 675-0004