Order number Battery-Pack SP (NiMH): **8713180**
Order number Battery-Pack SP (NiMH) with pin: **8713180A**
Order number Battery module SP (NiMH): **34520856**
Order number Battery module SP (NiMH) with pin: **34520856A**
Order number Battery service kit PSP: **34521360**
Order number Battery service kit ISP: **34521361**

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1 DESCRIPTION / BESCHREIBUNG

The below information is to be understood as an instruction for using the battery pack.

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3 USAGE OF THE BATTERY / ANWENDUNG DES AKKUMODULS

The battery pack is compatible with the following devices:

- Perfusor Space.
- Infusomat Space.
- SpaceStation with integrated SpaceCom.
- SpaceCover Comfort.

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8 ENVIRONMENTAL PROTECTION / UMWELTSCHUTZ

The battery pack is to be recycled as hazardous waste. Damaged batteries will be redeemed by B.Braun on request.

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9 CAUTION! / ACHTUNG!

- Hazard warning: an overheating of the battery pack, and an associated risk light burns, is possible if proper handling is not observed

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A guarantee of 12 months is provided with this battery pack. Typical lifespan of the batteries are 3-5 years. Lifespan may be shortened through regular use in areas with a high ambient temperature.

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5 STORAGE OF THE BATTERY / LAGERUNG DES AKKUMODULS

The following storage times are not to be exceeded.

Storage temperature (ambient)		Max. Storage Time
in Celsius	in Fahrenheit	
-20°C ... +35°C	-4°F ... +95°F	6 Months / approx. 180 days
-20°C ... +45°C	-4°F ... +113°F	3.5 Months / approx. 105 days
-20°C ... +55°C	-4°F ... +131°F	1.6 Months / approx. 50 days
-20°C ... +65°C	-4°F ... +149°F	1 Month / approx. 30 days

Recommendation: Charge batteries before storing for long periods of time. If the above maximum storage times are to be exceeded, a second charge during storage is recommended. Failure to comply with these recommendations may result in reduced battery lifespan.

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The operating temperature of the battery is between -10°C and +55°C (+14°F...+131°F).

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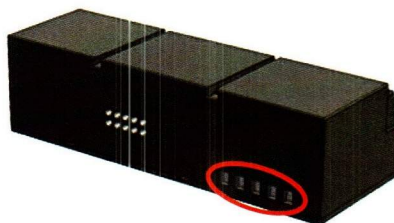
Darstellung:

Battery pack without pin (8713180):

Contact between battery pack and device occurs via 5 contact points on the battery pack (see markation in accompanying picture) and the contact array of the relevant device.

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Battery pack with pin (8713180A):

The new battery pack design includes a pin (fixation pin integrated into the housing). This pin improves the connection at the five contact points. The matching fixation hole is situated on the pictured contact array. This design is not reverse-compatible to versions without this pin or contact board.

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