D1 Carrying Case Instructions For Use

Introduction

The D1 carrying case is intended to contain and protect BeneHeart D1 (hereinafter called the D1) and accessories during transport in hospital or pre-hospital settings.

Content Management

Below is a recommended procedure to manage the contents of your carrying case. Perform the corresponding steps if the involved accessory has been purchased.

Step 1 Prepare the carrying case

1. Hook the belt to the metal rings on both sides of the carrying case, as indicated below:

2. Unzip the zipper of the carrying case to open it.

Step 2 Manage the ECG cable and D1

1. Connect the plug of the ECG cable to the ECG connector on the D1.
2. Put the D1 into the carrying case with the handle pointing to the opening on the top of the carrying case, as indicated below:

Front view

Side view


3. Carefully manage the ECG cable and put it to the left side of the D1.

Step 3 Manage the pads

1. Connect the pads cable to the pads connector on the D1.
2. Secure the cable with the velcro on the top of the carrying case as indicated below:

3. Put the pads package to the bag inside of the carrying case and secure it with the velcro.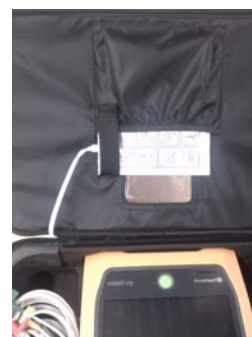

Step 4 Manage the battery and rescue bag

1. Check the items in the rescue bag. Make sure that a pair of scissors, a razor, a breathing mask, a towel or gauze, and two pairs of gloves are included.
2. Put the spare battery and rescue bag to the right side of the D1.

3. Zip the zipper of the carrying case.

Step 5 Confirm

1. Check that the carrying case is zipped up properly.
2. Check that the validation date on the pads package and the status indicator beside the Power On/Off button of the D1 are visible through the window of the carrying case.

Using the D1 with the Carrying Case

1. Lay the carrying case on a flat location.
2. Unzip the zipper of the carrying case.
3. Press the Power On/Off button to turn on the D1.
4. Take out the pads package, open it, and apply the pads on the patient as indicated. If necessary, process the application site for the pads with tools included in the rescue bag.
5. Follow the instructions of the D1.

For detailed information, refer to the Operators' Manual of the D1.

Cleaning

The carrying case can be cleaned by wiping with a soft, clean cloth dampened with water. Cleaning agents can be used to remove stubborn stains.

www.mindray.com

Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen 518057 P.R. China

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