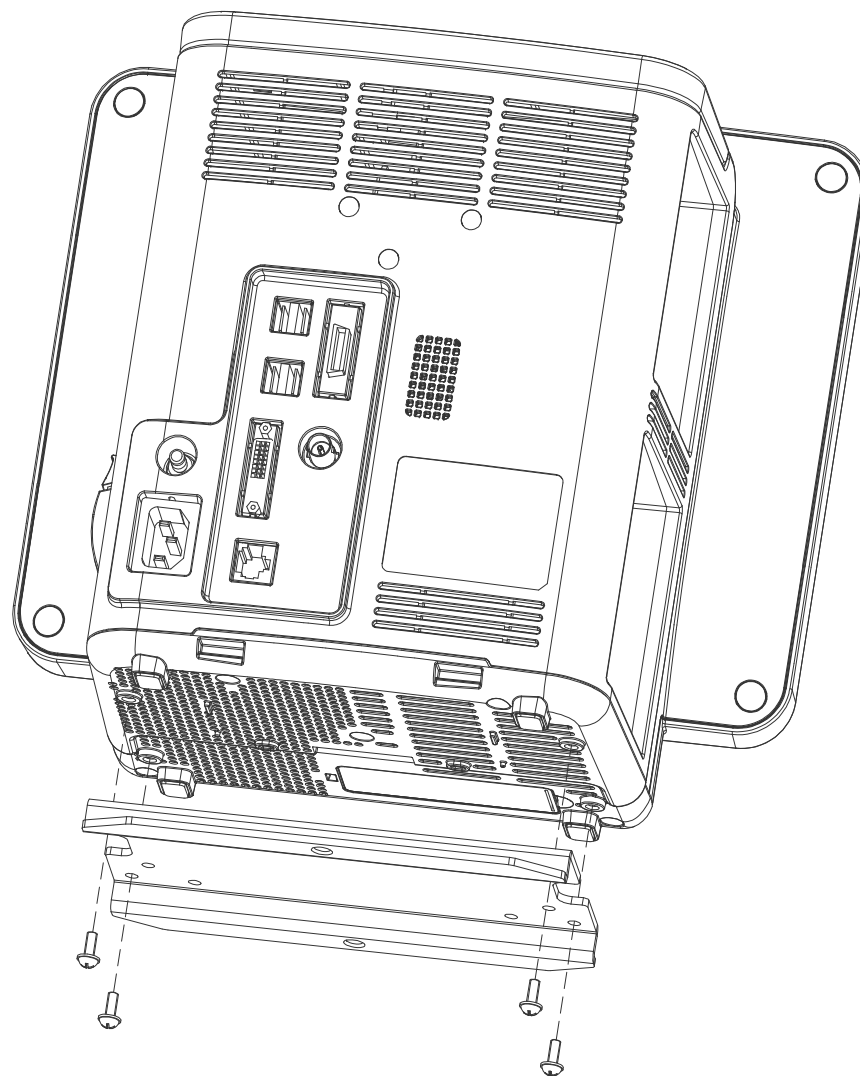
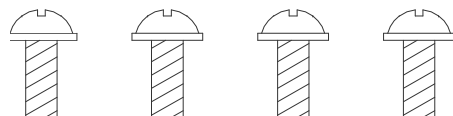
M4X10

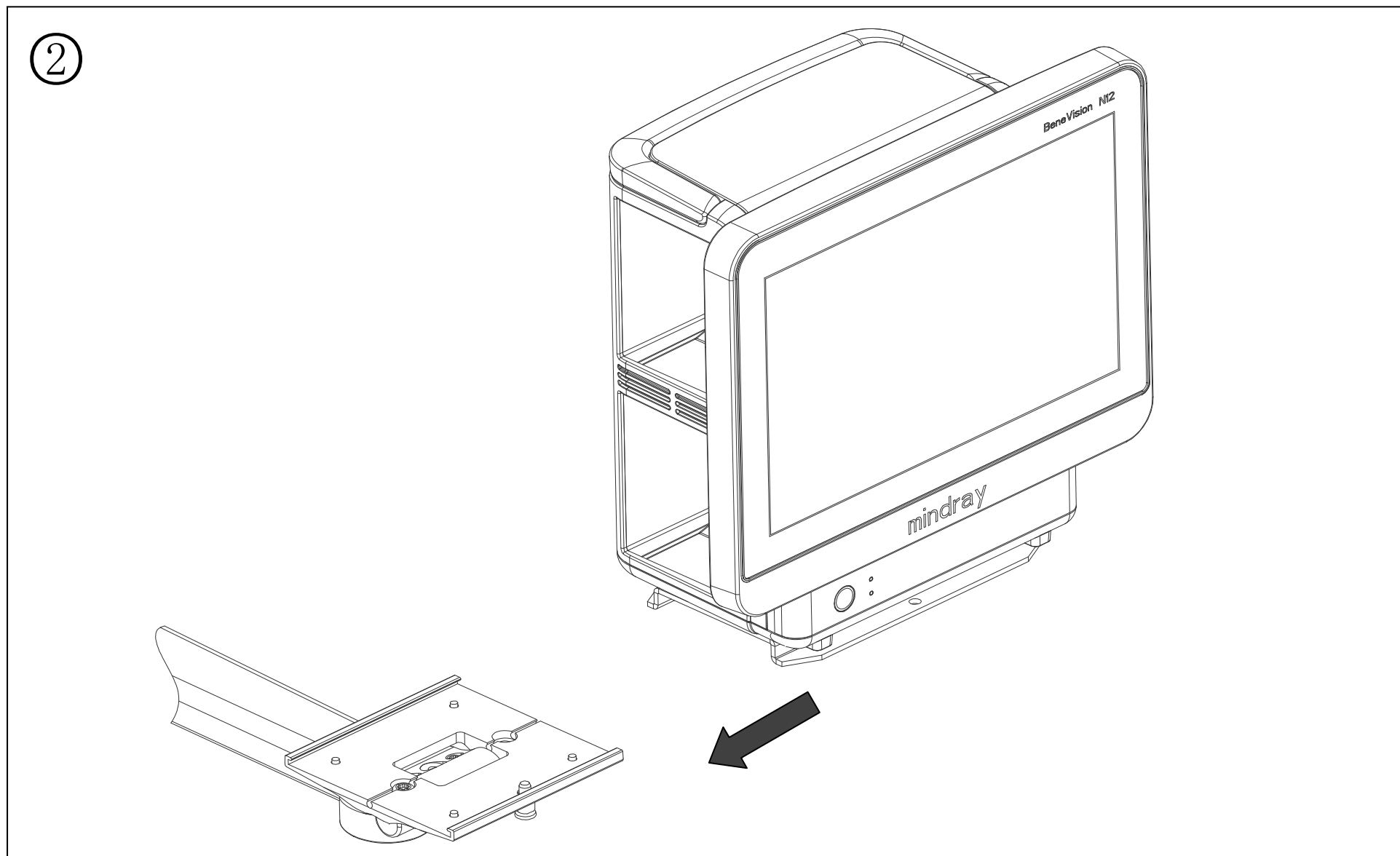
046-010865-00(1.0)

①

M4X10

Cleaning\Disinfection 清洁消毒

Cleaning\ Disinfection:

1. Clean the mounting assemblies with most mild, non-abrasive solutions commonly used in the hospital environment, for example diluted bleach, ammonia, or alcohol solutions).
2. Use a water-dampened cloth to wipe any cleaning agents on the arm immediately.
3. Dry the arm thoroughly after cleaning.

CAUTION:

- MR makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted devices or accessories, refer to the specific instructions.
- Damage caused by the use of unapproved substances or processes will not be warranted. It is recommended that you test the cleaning solution on a small area of the arm that is not visible to verify compatibility.
- The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene. Do not immerse or soak the bracket or wet its surface.
- Never use steel wool or other abrasive material to clean the mounting assemblies.
- Never submerge the arm, and do not allow liquids to enter the arm.

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