

Veta 3/Veta 5 Installation Guide


1. Install the Trolley

Insert the column into the trolley base; then, put the trolley down and tighten the screws.




2. Install the Main Unit

Lock four casters of the trolley. Then, place the main unit securely on the trolley by aligning three positioning columns under the main unit with three locating pins on the trolley, and tighten screws.

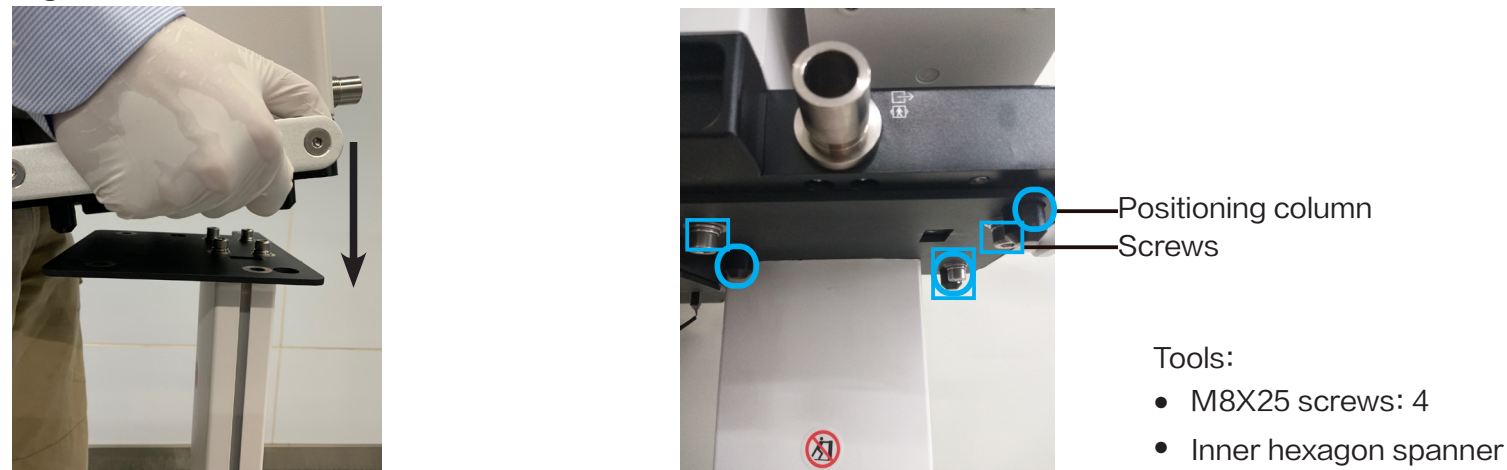


3. Install the Breathing Circuit (Veta 5 Only)

- Align the guiding post slots of the breathing system with the guiding post of the main unit circuit connector, and fix the breathing circuit with screws.
- Fix the fastener under the supporting board and insert the inner hexagon spanner into it.
- Insert the microswitch cable plug into the slot.

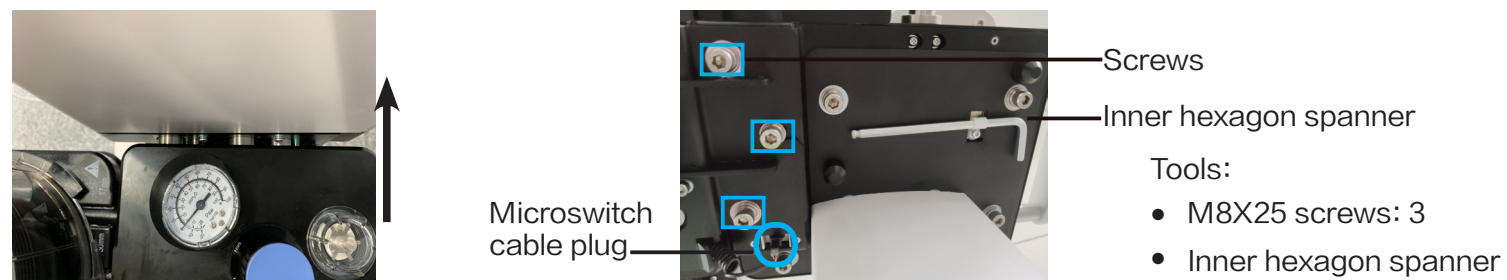


4. Install the CO2 Absorbent Canister

Rotate the CO2 absorbent canister counterclockwise to fix it.

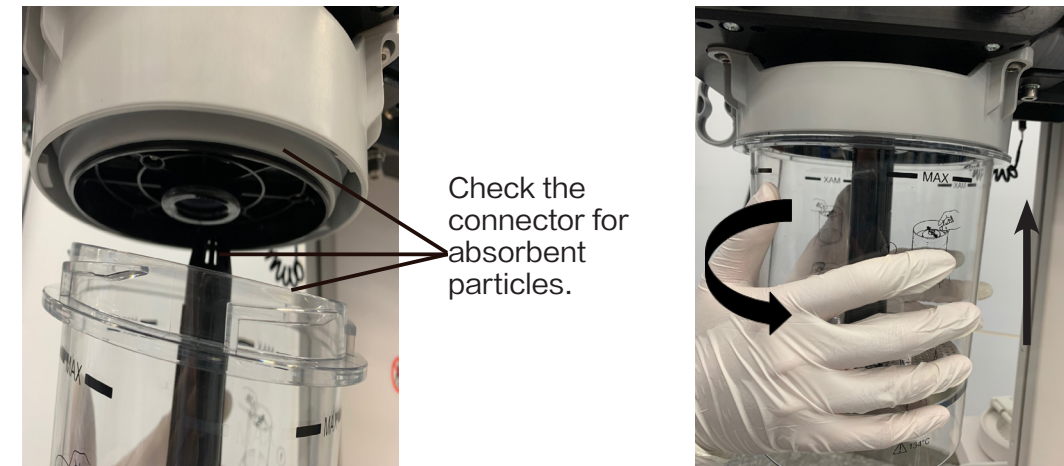


5. Install the Bellows (Veta 5 Only)

- Attach the bottom ring of the bellows to the bellows base and ensure that the bellows is tightly connected to the base.
- Align the bellows housing bayonet with the slots, and rotate it clockwise firmly.

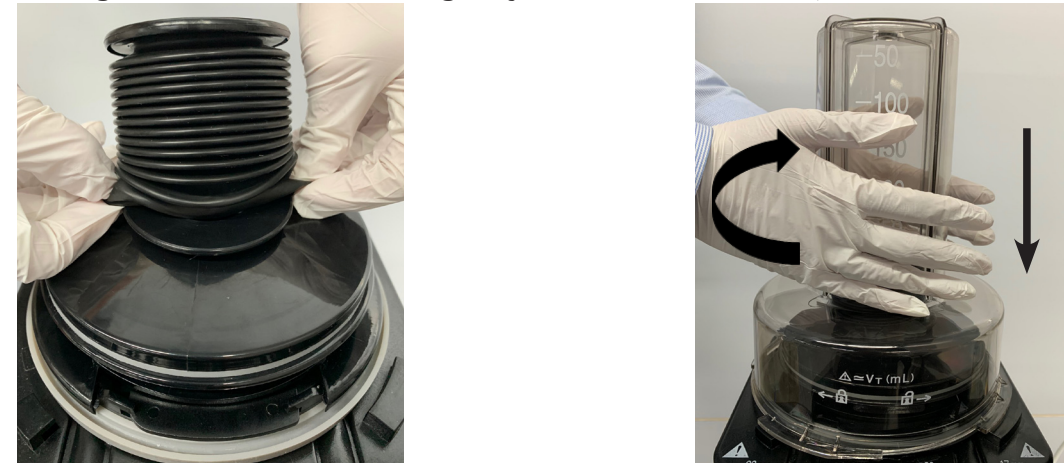

6. Install the Basket


Hang the basket on the handrail of the machine directly or install it on the column.

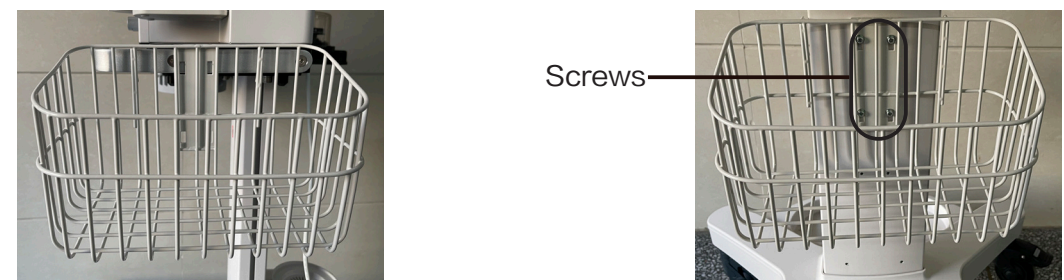


Veta 3/Veta 5 Installation Guide

7. Install the Oxygen Generator Tray and Connector (Optional)

- Fix the oxygen generator tray assembly to the column and trolley by using screws and the fixing plate respectively.
- Rotate the oxygen generator connector clockwise into the gas supply inlet and connect the oxygen generator with a tube.

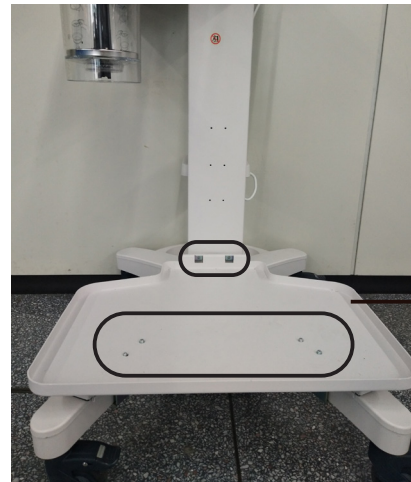

Tray assembly

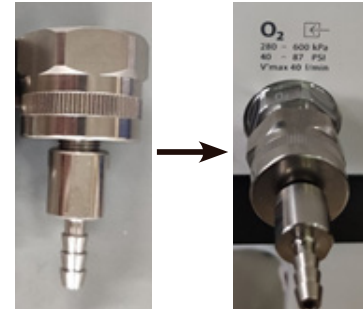

Tools:

- M4X12 screws: 6
- Phillips screwdriver

9. Install the Anesthesia Gas Filter Canister

- Place the anesthesia gas filter canister (self-prepared) on the weigher or the tray by aligning the groove.
- Use the silicone tube to connect the gas filter canister with the AGSS inlet pipeline connector.



AGSS inlet pipeline connector

Silicone tube


Anesthesia gas filter canister

8. Install the Tray or Weigher (Optional)

- Insert the tray or weigher (Veta 5 Only) into the screw arbor and tighten the screws.
- Connect the weigher cable with the communication interface (SP).
- Use three silicon plugs to attach the cable to the column and fix the cable on the back of the machine.



Screws

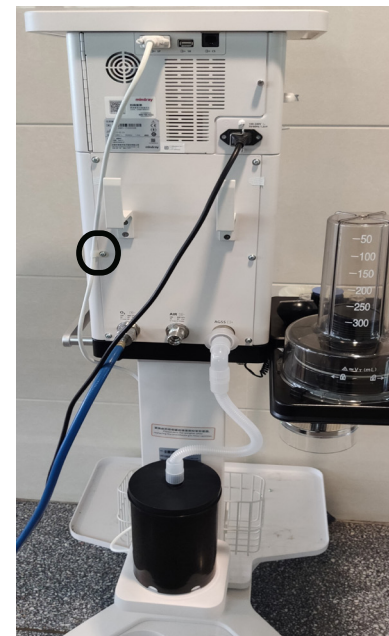

Tools:

- M4X12 screws: 4
- Phillips screwdriver

10. Connect the AGSS System or Passive Scavenging Pipe (Optional)

- Rotate the two stop pins into the AGSS support rail and fix the AGSS support rail into the side of column with screws.
- Insert the AGSS assembly into the AGSS support rail and rotate the knob tightly. Use the AGSS gas supply hose to connect the waste gas outlet with the AGSS assembly inlet.
- Connect the waste gas outlet with the passive scavenging pipe when using the passive scavenging method.



Screws

AGSS support rail

Stop pin



AGSS gas supply hose



Knob



Passive scavenging pipe

Tools:

- M4X12 screws: 2
- Phillips screwdriver



046-022515-00(1.0)