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INSTRUCTION FOR USE

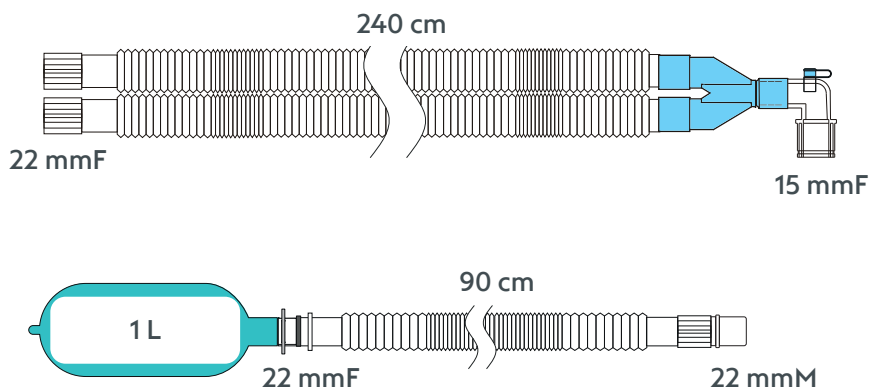
NOVO BC Extend Kids 240SE

Breathing Circuit

REF N20424031

NOVO®

IFU_Rev. 03_202507

Manufactured in Cleanroom

Patientenzielgruppe
Patient target group

Beatmete Patienten / Anästhesie
Ventilated patients / anaesthesia

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EN | INSTRUCTION FOR USE

Disposable extendable breathing circuit for children for conducting respiratory gases between anaesthesia machine or ventilator and patient, for single use only.

1. NOTICE ON REPORTING REQUIREMENTS

Please inform the manufacturer and the competent authority of the Member State in the event of any serious incident that has occurred in relation to the device.

2. Application by healthcare professionals or users trained by healthcare professionals.

CONTRAINDICATIONS

The device is used only as directed. Improper use of it will harm the patient.

WARNING NOTICES

- Use the breathing circuit only with components contained in the set.
- The medical device must not be reused or reprocessed.
- Modifying the medical device may cause damage and endanger the patient.
- Obstructions, damage, and foreign matter can lead to malfunction. Check the breathing set for damage prior to use.
- Perform a selftest of the basic device including a leakage test after the breathing circuit has been completely installed and before use on the patient.
- If too much condensate accumulates, a partial or complete blockage of the breathing circuit can occur. Regularly check for condensate.
- Adding further components and using incompatible components may adversely affect the performance of the ventilator.
- If includes; A Luer Lock connector must only be used for gas monitoring.

Novo Klinik-Service GmbH
Technologiepark West – Haus 15 |
Zum Frenser Feld 1
50127 Bergheim | Germany

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